

Gator Instrument Clips™

Instructions for Use and Product Information:

Product Brand Name:	Gator Instrument Clip™ and Tip Guard
Generic Name:	Instrument Clip
Product Order #s:	GIC190-5, GIC190-10, GIC270-5, GIC270-10, GTG-A-500, GTG-A-2000
Purpose of Product:	Use to keep instrument jaws open during steam sterilization.
Product Range of Application:	GIC190 – use on ring-handled scissors GIC270 – use with ratcheted ring-handled instruments GTG-A-2000 – Used on delicate scissor tips
Product Description and Key Product Specifications:	A steam sterilization compatible clip designed to hold instrument jaws in the open position during sterilization.
Instructions for Use:	<ol style="list-style-type: none"> Slide closed scissor tips into the Gator Guard pouch. Attach each of the Gator Instrument Clip ringlets onto each of the instrument's ring handles. Insert the instrument with the Gator Instrument Clip applied, into an appropriately-sized peel pouch, wrapped tray or closed containers system. Follow the facility's policies for instrument sterilization. Discard after use.
Product Disposal:	Single-Use and disposable as municipal waste.
Latex Information:	These products are latex-free.
X-Ray Detectable:	These products are x-ray detectable.
Adhesives:	N/A
Sterilization	Validated for use in Pre-Vacuum Steam Sterilization.

Sterilization Validation: Steam Sterilization			
Cycle Type	Temperature	Minimum Time Full Cycle	Minimum Validated Dry Time
Pre-Vacuum	132° C (270° F)	4 Minutes	20 Minutes

Product shipping & storage Information on the next page

Gator Instrument Clips™

Instructions for Use and Product Information continued:

PRODUCT SHIPPING & STORAGE INFORMATION	
Shipping Conditions:	N/A
Requirements:	N/A
Storage Conditions:	N/A
Packing Conditions:	N/A
Shelf Life:	N/A