




Introducing a patent pending Bumper designed for the operating room table to provide patient safety and surgeon relief. Cushion Muscles From Operating Room Table known as ...

# the **COMFORT**

The RC Medical LLC COMFORT\* is a single use device that is packaged non-sterile. It is used in the operating room only in conjunction with the lateral rail of the operating room table which is considered a non-sterile environment. The COMFORT protects the patient from injury secondary to the lateral rail and cushions the surgeon while leaning against the operating room table.

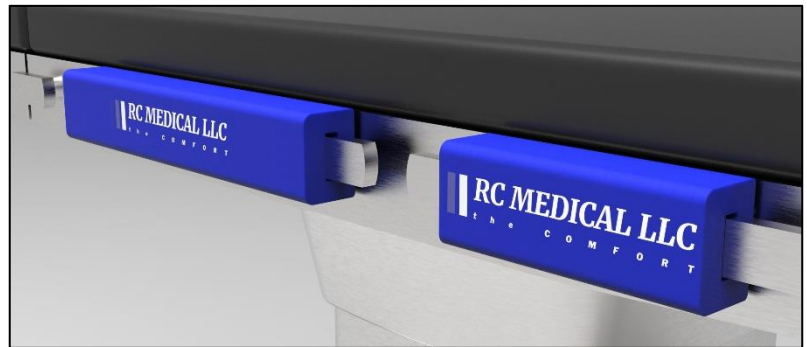

## **Using the COMFORT**



- The COMFORT is a cushion, made from a synthetic rubber extrusion foam, designed to protect personnel in the operating room from the protruding lateral steel rail located on the operating room table below the sterile field (OR bed).
- Place the COMFORT onto the rail on the surgeon's operating side of the table prior to the patient entering the room.
- If desired, place an additional COMFORT onto the opposite side of the operating room table for the surgical assistant.
- The COMFORT can be repositioned by a non-sterile attendant at any time during the procedure while maintaining the sterile field integrity by approaching the COMFORT from underneath the sterile drape covering the table (OR bed).

There are two different sizes of COMFORT rail cushions, the COMFORT 6 and the COMFORT 12.

- The COMFORT 6 package consist of 4 EA 6" COMFORT rail guard pads. They are sold in cases of 18 packages. Case order number is 30008. The physical size of the case is 12" L x 12" W x 10.5" H.
- The COMFORT 12 package consist of 2 EA 12" COMFORT rail guard pads. They are sold in cases of 18 packages. Case order number is 30009. The physical size of the case is 12" L x 12" W x 10.5" H.

The **COMFORT** in position on the operating room table rail.



Herzog Surgical  
5901 Rosebud Lane, #200  
Sacramento, CA 95841

Website  
E-mail  
Phone

[www.hertzogsurgical.com](http://www.hertzogsurgical.com)  
[sales@hertzogsurgical.com](mailto:sales@hertzogsurgical.com)  
(916) 334-1288